



## UnoDose™ Applicator, 0.25mL/click, 42mL, White

**Description:** From top to bottom and inside out, the UnoDose Metered-Dose Topical Applicator is designed to enhance provider efficiency and patient ease of use – all while keeping costs competitive. The end result is an improved bottom line for pharmacy businesses of all sizes.

The single-use UnoDose Metered-Dose Applicator repeatably dispenses a metered volume of therapeutic medication for application to the skin. It also functions as a compounding container for use with electronic mortar and pestle systems or MAZ mixers using a specialized adapter.

### Features for Providers:

- In-container machine compounding, no transfer required
- Multiple exit holes for reliable priming
- Fast prime feature quickly removes air with no repetitive motion, saving time
- Open, screw-free chamber for easy filling
- Disposable mixing paddles for fast cleanup
- Registered as an FDA Class I medical device

### Features for Patients:

- Allows cream application without hand contact
- Multiple exit holes for uniform topical application
- Audible, tactile and visual metered dose indicators for accurate dosing
- Locking cover secures against accidental spills
- Clearly visible reorder indicator

### Technical Specifications:

USA | Canada | Australia | International  
medisca.com

<b>Dose size</b>	0.25 mL per click
<b>Maximum usable volume</b>	42 mL
<b>Residual volume</b>	1.5 mL
<b>Dimensions, w x h (cm/in)</b>	/ 1.87 x 4.23
<b>Medicament contacting materials</b>	Polyolefins free of phthalates and BPA
<b>UV protection</b>	UV-inhibited materials used for the medicament container

### Packaging Specifications:

- All pack sizes include one priming tool and patient instructions. Mixing lids and paddles sold separately.

### Associated Products:

- 6006 – UnoDose™ Mixing Lid
- 6005 – UnoDose™ Mixing Paddle
- 6649 – Samix® Shaft, 15 – 100 mL
- 5793 – MAZ® UnoDose™ Mixing Adapter

**USA | CANADA | AUSTRALIA PACIFIC | EUROPE**

[WWW.MEDISCA.COM](http://WWW.MEDISCA.COM)

Please note that the information contained herein is for informational purposes only. The information is based on recognized compendia, manufacturers' instructions and/or other authoritative sources and texts. MEDISCA takes no responsibility for the validity or accuracy of this information. Customer MUST refer to USP, FDA, Health Canada, TGA and/or their State Board to ensure they comply with the specific regulations and ensure that appropriate procedures are in place.


**STEP 1:**

**Transfer** the desired amount of compounded cream into the UnoDose™ Applicator barrel using a spatula, ointment jar, or filling system.


**STEP 2:**

For best results, remove air during filling. Start with a small amount to **fill the corners** in the bottom of the barrel. **Tap the bottom** of the applicator on your palm or a countertop to settle the cream and remove air.


**STEP 3:**

**Add the remaining cream** in several steps while continuing to **tap the bottom** of the applicator to settle the cream and remove air. Leave the top of the cream flat and smooth.


**STEP 4:**

Put the applicator cap on the barrel and **push downward** to snap into place. The applicator cap will butt up against the ring on the barrel. **Tap the bottom** again and confirm that no air remains in the cream.


**STEP 5:**

**Prime to remove air.** For a fast prime, **use the prime tool supplied in each box.** **Push slowly** until cream just comes out the holes, or the tool stops.

**NOTE:** The short end of the tool leaves 32 usable millimeters in the applicator, the long end leaves 17.


**STEP 6:**

Finish priming by **turning the dosing knob** several clicks until cream exits all holes smoothly and equally.

**NOTE:** Be sure to leave the knob in a "lock" position, and not between clicks.


**STEP 7:**

**Wipe away the cream** from the applicator cap, then **lock the protective cover** in place with a light push and clockwise turn. You will hear and feel a click.


**STEP 8:**

Confirm the correct volume on the scale by **reading at the blue line.**

**NOTE:** The scale shows the usable volume and clicks remaining, and excludes the residual volume of 1.5 millimeters.


**STEP 1:**

**Remove** the protective cover with a light counterclockwise twist and pull while holding the barrel.


**STEP 2:**

Turn the dosing knob in the direction marked for the number of prescribed clicks.

**NOTE:** Be sure to leave the knob in a "lock" position, and not between click positions.


**STEP 3:**

**Confirm** that cream has exited the holes of the applicator cap.


**STEP 4:**

**Apply** the cream to the skin absorption site. **Rub** with the applicator cap until the cream is evenly spread (per physician or pharmacist direction). **Replace** the protective.


**STEP 5:**

**Check** the scale at the blue line for the number of clicks remaining. **Refill** your prescription as instructed by your pharmacy.